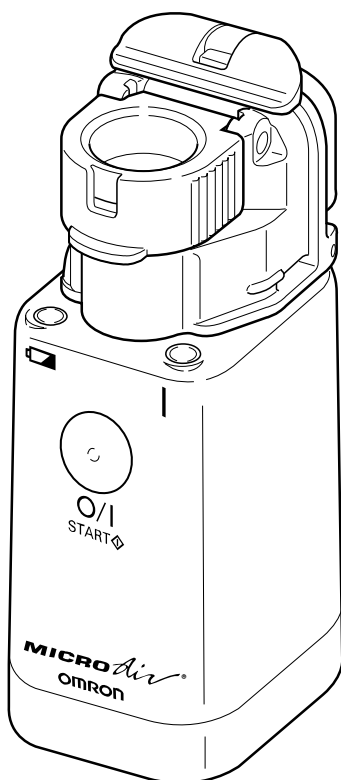
# OMRON

## INSTRUCTION MANUAL

### MICRO *Air*<sup>®</sup>

## VIBRATING MESH NEBULIZER

Models **NE-U22V** and  
**NE-U22VAC**

**Caution:** Federal law restricts this device to sale by or on the order of a physician.

# TABLE OF CONTENTS

## Before Using the Device

Introduction.....	3
Safety Information.....	4
Components and Functions.....	5-6
Accessories and Replacement Parts.....	6
Unit Assembly.....	7-8

## Operating Instructions

Battery Installation.....	9
Using the Optional AC Adapter.....	10
Filling the Medication Bottle.....	11
Selecting the Nebulization Mode.....	12
Positioning the Device.....	13

## Care and Maintenance

Cleaning after Each Use.....	14
Daily Disinfecting.....	15
Caring for the Device.....	16

## Troubleshooting

Troubleshooting Guide.....	17
Warranty Information.....	18
Specifications.....	19

The illustrations shown in this Instruction Manual are image drawings.

# INTRODUCTION

Thank you for purchasing the Omron MICROAir® Vibrating Mesh Nebulizer.

*Fill-in for future reference*

DATE PURCHASED: \_\_\_\_\_

SERIAL NUMBER: \_\_\_\_\_

*Staple your purchase receipt here.*

Omron Healthcare has advanced the development of electronic nebulizer technology with the introduction of the MICROAir® Vibrating Mesh Nebulizer. Focusing on patient convenience and compliance as its goal, this device offers ultimate portability for wherever you go and revolutionary vibrating mesh technology that provides a precise, powerful and effective treatment every time.

NE-U22V/NE-U22VAC MICROAir® comes with the following components:

- Main Unit
- Unit Cover
- Medication Bottle
- Mesh Cap
- Mask Adapter
- Mouthpiece
- Storage Case
- Instruction Manual
- Instructional Video

**NOTE: Model NE-22VAC includes the AC Adapter.**

*This unit is a medical device. Operate this unit only as instructed by your physician and/or respiratory therapist.*

**SAVE THESE INSTRUCTIONS**

# SAFETY INFORMATION

To assure the correct use of the product, basic safety measures should always be followed including the precautions listed in the instruction manual.

## SAFETY ICONS USED IN THIS INSTRUCTION MANUAL

 <b>CAUTION</b>	Incorrect use or handling of the device may result in bodily harm or material damage.
	Risk of electrical shock.
	Important Notice.

- ❗ Read all the information in the instruction book and any other literature included in the box before using the unit.
- ❗ Clean and disinfect the Medication Bottle, Mesh Cap, Mask, Mouthpiece and Mask Adapter before using the device for the first time after purchase, if the device has not been used for a long period of time, or if the same device is used by more than one person.
- ❗ For type, dose, and regimen of medication follow the instructions of your healthcare provider.
- ❗ Provide close supervision when this device is used by, on, or near children or compromised individuals.
- ❗ Keep the device out of the reach of infants and children. The device may contain small pieces that can be swallowed.
- ❗ Changes or modifications not approved by Omron Healthcare will void the user warranty. Do not disassemble or attempt to repair the unit or components.
- ❗ Operate the unit only as intended. Do not use the device for any other purpose.

# COMPONENTS AND FUNCTIONS

## Electrode

Power conductor from the Main Unit to the vibrator on the Medication Bottle

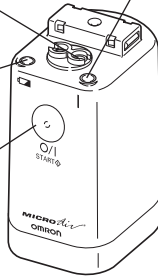
## Battery Low Indicator

An orange light blinks when the batteries are worn

## ON/OFF Button

Turns the power for the Main Unit on and off

## Main Unit

## Power Indicator

The green light shows the power is on

## Bottom of Main Unit

### Battery Cover

Push lever to remove cover

### Electrode

Power conductor from the Main Unit to the AC Adapter Connection Stand

## Bottom of Main Unit

### Bottle Cap Locking Lever

Opens the Medication Bottle for cleaning

### Medication Bottle

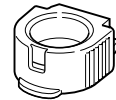
Holds maximum capacity of 7mL for treatment

### Medication Port

Add the medicine to the bottle

## Mesh Cap

Metal Alloy Mesh creates high efficiency aerosol

## Mask Adapter

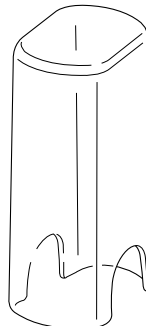
Holds the Mouthpiece or Mask securely on the device

## Mouthpiece

Patient Interface

## Main Unit Cover


Protects the Main Unit with the attached Medication Bottle and Mesh Cap during storage

# COMPONENTS AND FUNCTIONS

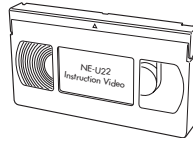
## Instruction Manual

## Instruction Video


## Storage Case

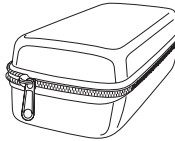Case holds the Main Unit, Mask Adapter and Mouthpiece

## Optional Accessory
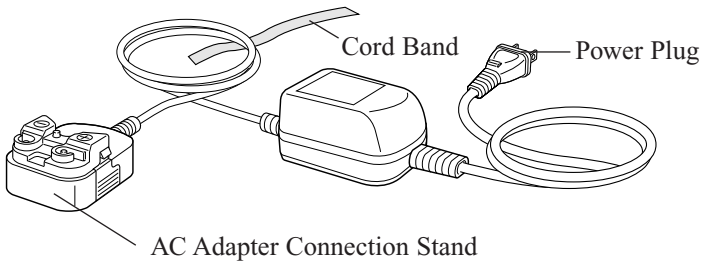
### AC Adapter Model No. U22-5

## Optional Accessory

### Child's Mask Model No. 9921

Patient Interface for pediatric use

## Replacement Parts

	Model No.
Battery Cover	U22-8
Main Unit Cover	U22-9
Mask Adapter	U22-2
Medication Bottle	U22-3
Mesh Cap	U22-4
Mouthpiece	U22-1
Storage Case	U22-7

# UNIT ASSEMBLY

**You must clean and disinfect all removable parts before using the device.**

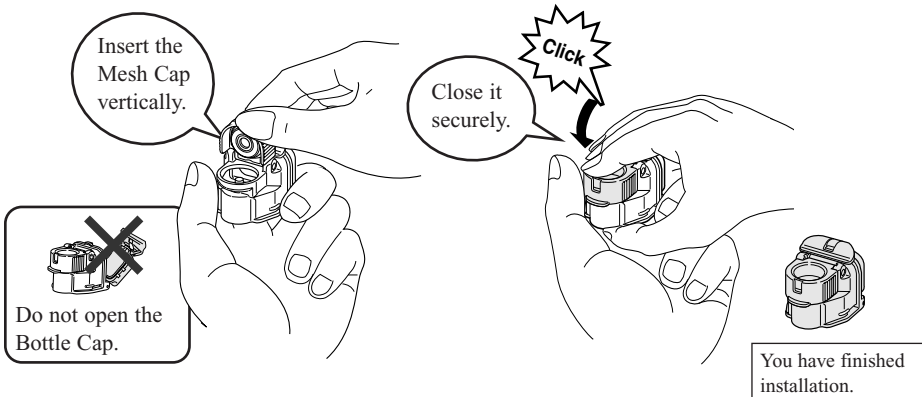
- ❗ For directions on cleaning and disinfecting go to page 14 and 15 under Care and Maintenance.

**The device must be assembled before it is used.**


## General Information

- Components may fit tightly since they are made to prevent the medication from leaking.
- Hold the device securely with both hands.
- Install the parts securely. You may hear a click sound as you install some of the parts.

### 1. Attach the Mesh Cap to the Medication Bottle.


### 2. Attach the Medication Bottle to the Main Unit.
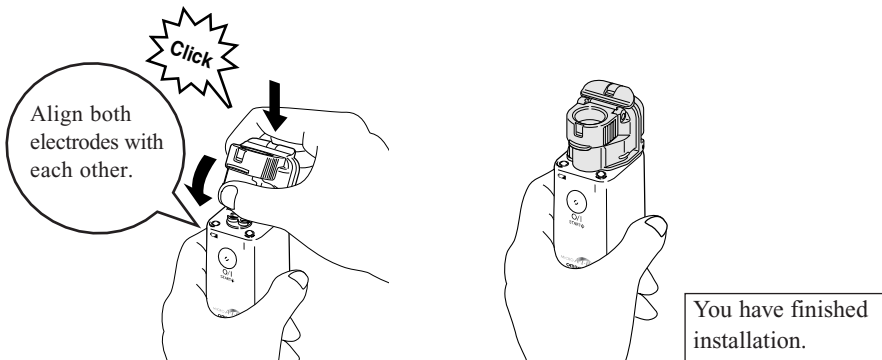
# UNIT ASSEMBLY

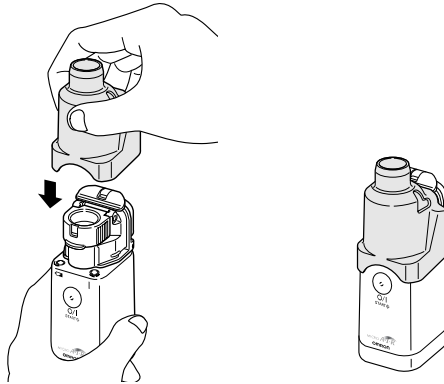
3. Attach the Mask Adapter to the Main Unit.

You have finished installation.


4. Attach the Mouthpiece or the Child's Inhalation Mask to the Mask Adapter.

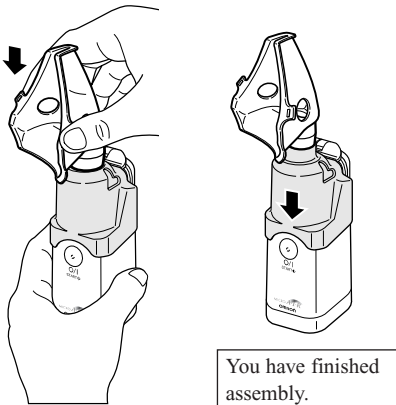• How to attach the Child's Inhalation Mask

You have finished assembly.

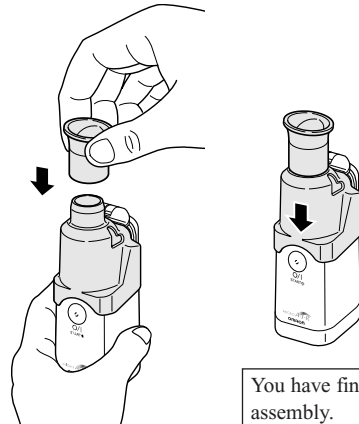• How to attach the Mouthpiece

You have finished assembly.

# BATTERY INSTALLATION

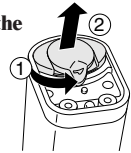
This device operates using two (2) AA alkaline batteries or two (2) NiMH AA rechargeable batteries.

- ❗ Do not install worn and new batteries together.  
Do not use different types of batteries together.  
Remove the batteries if the device will not be used for three months or longer.

## 1. Remove the Battery Cover.

- ① Rotate the Battery Cover lever in the direction of the arrow as shown in the illustration.
- ② Remove the Battery Cover. The Battery Cover may appear to be tight fitting since designed to prevent fluids from getting into the device.

Bottom of the  
Main Unit


## 2. Insert the batteries.

Correctly align the polarities (+ and -) with the battery indication marks on the device.

## 3. Replace the Battery Cover.

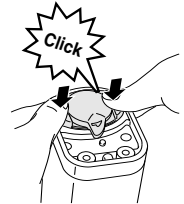Use your thumbs to push on both ends of the Battery Cover. Press down firmly until you hear both tabs click into place.

## Battery Life and Battery Replacement

### Alkaline Batteries

- The device can be used for approximately 8 days if operating for 30 minutes a day.
- The Battery Low Indicator (orange light) flashes to signal the batteries are low. Replace both batteries with new ones.
- The Battery Low Indicator (orange light) turns on to signal the batteries are worn out. The device will not nebulize. Immediately replace both batteries with new ones.

### NiMH Rechargeable Batteries

- The device can be used for approximately 8 days if operating for 30 minutes a day when the batteries are fully charged.
- The Battery Low Indicator (orange light) flashes to signal the rechargeable batteries have little or no residual power remaining. If the device will not nebulize immediately recharge the batteries.
- Recharge the batteries using a commercially available battery charger suitable for the batteries used in the device.
- The AC Adapter does not function as a battery charger.

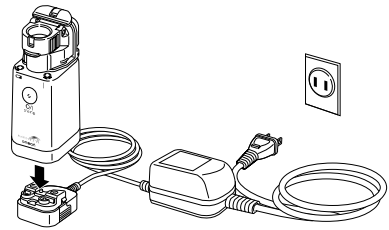
# USING THE OPTIONAL AC ADAPTER

The device is designed not to draw power from the batteries when the AC Adapter is used. The AC Adapter is not a battery charger.

## To connect the AC Adapter to the Main Unit


1. Place the Main Unit on the AC Adapter Connection Stand as shown in the illustration below.

**NOTE:** It will click and lock to the stand.

2. Insert the AC Adapter Power Plug into a 120V electrical outlet.

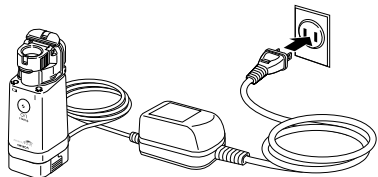 Do not insert or remove the Power Plug from the electrical outlet with wet hands.


Connection is completed.

## To remove the AC Adapter from the Main Unit

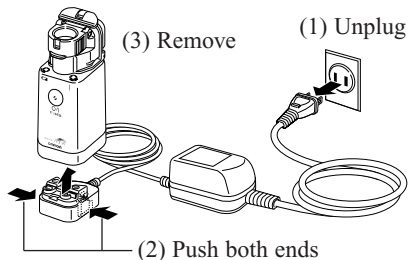- (1) Disconnect the AC Adapter Power Plug from the electrical outlet.
- (2) Push on both sides of the Connection Stand to unlock the Main Unit.
- (3) Remove the Main Unit.

 Disconnect the Power Plug from the electrical outlet when not in use.

# FILLING THE MEDICATION BOTTLE

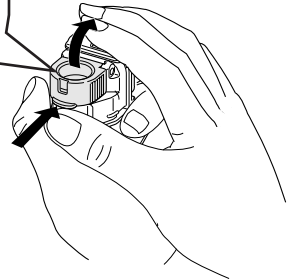
Remove the Mouthpiece or Mask and the Mask Adapter from the Main Unit.

## 1. Open the Mesh Cap.

Hold the device securely in your hand.


Lift up the mesh cap while pressing your thumb upward on the brim.

## 2. Fill the Medication Bottle.

Be careful to prevent the Mesh Cap from closing as shown in the illustration.

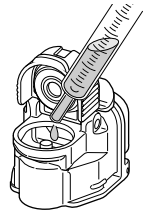The maximum capacity of the Medication Bottle is 7mL.

### Caution

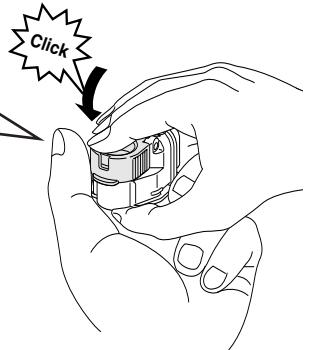
To prevent damage to the device, add the medication slowly. Do not allow the medication to overflow the medication port.

## 3. Close the Mesh Cap.

### Caution

To prevent damage to the device, engage the Mesh Cap correctly in place. If the Mesh Cap is not properly closed medication will leak.

Check that the Mesh Cap is closed.

## 4. Attach the Mask Adapter to the Main Unit.

Attach the Mouthpiece or the Child's Inhalation Mask to the Mask Adapter.

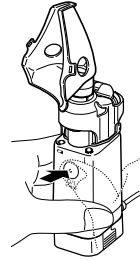
**NOTE:** For instructions on attaching the Mask Adapter and Mouthpiece return to Unit Assembly on Page 8.

# SELECTING THE NEBULIZATION MODE

This device operates in the Continuous Nebulization Mode or in the Manual Nebulization Mode.

## • Continuous Nebulization Mode

To start the device using the continuous nebulization press and hold the ON/OFF Button down for 1 second. Press the ON/OFF Button again to stop nebulization.


Press and hold the ON/OFF Button down with your finger for 1 second.

## • Manual Nebulization Mode

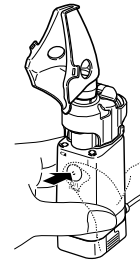In the Manual Nebulization Mode the device will nebulize only when you press and hold the ON/OFF Button down. You can inhale on demand using this mode.

To start the device using manual nebulization press and hold the ON/OFF Button down for at least 2 seconds.

Press and hold the ON/OFF Button to start nebulization.  
Press and hold the ON/OFF Button to stop nebulization.

Press and hold the ON/OFF Button down with your finger for 2 seconds.


**NOTE:** The power indicator (green light) illuminates during nebulization.

# POSITIONING THE DEVICE

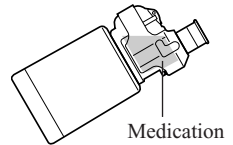Operate the device in the Continuous Nebulization Mode or in the Manual Nebulization Mode. Operate this device only as instructed by your healthcare provider and/or respiratory therapist.

1. Slightly tilt the Main Unit towards yourself to immerse the Vibrating Mesh Cap in the Medication.

**NOTE:** If the vibrator is not immersed in the medication, the device will not nebulize.

2. Start inhalation in a relaxed position.
3. Place your lips lightly around the mouthpiece.  
If using the Child's Inhalation Mask position the mask lightly against the face.
4. Start the treatment as directed by your healthcare provider.
5. Press the ON/OFF Button to turn off the device when finished with your treatment.

**NOTE:** The device has a built-in timer to turn off the power approximately 30 minutes after the power is turned on. When the AC Adapter is used, remove the power plug from the electrical outlet.

# CLEANING AFTER EACH USE

Following the cleaning instructions after each use will prevent any remaining medication in the bottle from drying, adhering to the mesh cap, and resulting in the device not nebulizing effectively.

## **Wash the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask Adapter after each use.**

1. Remove the Mouthpiece or Mask and the Mask Adapter from the Main Unit.
2. Remove the Medication Bottle from the Main Unit.
3. Open the Medication Bottle and discard any remaining medication.
4. Attach the Medication Bottle to the Main Unit. Open the Mesh Cap.
5. Pour a small amount of distilled water into the Medication Bottle and close the Mesh Cap.
6. Turn on the device to nebulize the distilled water for 1 to 2 minutes to remove residual medication from the mesh holes.
7. Turn off the device and remove the Medication Bottle from the Main Unit.
8. Remove the Mesh Cap from the Medication Bottle and discard any remaining distilled water from the Medication bottle.
9. Rinse the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask Adapter with distilled water.

### **Caution**

#### **To prevent damage to the device:**

- Do not rinse or immerse the Main Unit in any liquid.
  - Do not wash or rinse any of the parts under strong running water.
  - Do not touch the mesh with your hand or any other object.
10. Gently wipe off excess water with a soft clean cloth or allow the parts to air dry in a clean environment.
  11. Assemble the device. Store the device in the Storage Case or in a clean environment.

# DAILY DISINFECTING

**Disinfect the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask Adapter after the last treatment of each day.**

1. Make a solution of 1 part white vinegar and 3 parts distilled water.

** Caution**

Do not use household bleach. The mesh will rust.

2. Lift open the Mesh Cap and pour a small amount of disinfecting solution into the Medication Bottle.
3. Turn on the device to nebulize the disinfecting solution for 1 to 2 minutes.
4. Turn off the device and remove the Medication Bottle from the Main Unit.
5. Remove the Mesh Cap from the Medication Bottle and discard any remaining disinfecting solution from the Medication Bottle.
6. Soak the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask Adapter in the disinfecting solution for 10 to 15 minutes.
7. Rinse the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask Adapter in distilled water.

** Caution**

To prevent damage to the device:

- Do not rinse or immerse the Main Unit in any liquid.
  - Do not wash or rinse any of the parts under strong running water.
  - Do not touch the mesh with your hand or any other object.
8. Gently wipe off excess water with a soft clean cloth or allow the parts to air dry in a clean environment.
  9. Assemble the device. Store the device in the Storage Case or in a clean environment.

# CARING FOR THE DEVICE

To keep your device in the best condition and protect the unit from damage follow these directions:

- Do not subject the device or the components to any strong shocks such as dropping the device on the floor.
- Do not store the device in extreme hot or cold temperatures, high humidity or under direct sunlight.
- Remove the batteries if the device will not be used for three months or longer. Always replace all the batteries with new ones at the same time. Do not use different types of batteries together.
- Use only authorized Omron Parts and Accessories.

## Cleaning the Main Unit

To clean the casing of the main unit moisten a soft cloth with water or a mild detergent. Wipe the casing and immediately dry using a soft clean cloth.

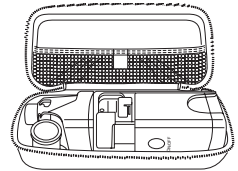
### **Caution**

#### **To prevent damage to the device:**

- Do not rinse or immerse the Main Unit in any liquid.
- Do not clean the Main Unit using abrasive cleaners or any type of chemical.
- Do not allow any moisture to contact the electrodes or the AC Adapter Jack on the Main Unit or AC Adapter Connection Stand.

## Carrying the device in the Storage Case

- Assemble the device by attaching the Mesh Cap and the Medication Bottle to the Main Unit. Place the Main Unit Cover on the device.
- Position the device in the Storage Case as shown in the illustration.
- The Mouthpiece and Mask Adapter can be easily placed in the Storage Case to carry with the Main Unit.


### **Caution**

#### **To prevent damage to the device:**

- Do not carry the Medication Bottle filled with medication or distilled water.
- Do not put the AC Adapter in the Storage Case.

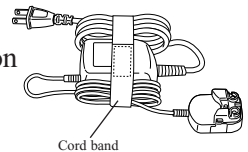The Storage Case is not intended to carry the optional AC Adapter.

To carry the AC Adapter bundle the power cord of the AC Adapter and fasten it to the Main Unit of the AC Adapter with the cord band as shown in the illustration.

# TROUBLESHOOTING GUIDE

PROBLEM	CAUSES AND SOLUTIONS
<p>The power indicator does not illuminate.</p>	<ul style="list-style-type: none"> <li>Be sure the AC Adapter is securely plugged into the electric outlet. Make sure the connection stand is correctly attached to the Main Unit.</li> <li>Follow the diagram for proper placement of batteries.</li> <li>Replace both worn batteries immediately.</li> <li>Recharge NiMH batteries with a commercially available charger.</li> </ul>
<p>The power indicator is illuminated but the unit does not nebulize.</p>	<ul style="list-style-type: none"> <li>Slightly slant the Main Unit towards you ON/OFF button pointing down.</li> <li>Be sure Mesh Cap is closed securely.</li> <li>Be sure that there is no liquid on top of the Mesh Cap. If visible remove with a light touch using a soft cloth to absorb.</li> <li>Fill the Medication Bottle with the proper amount of prescribed medication. Max is 7ml.</li> <li>Check for any liquid that may have collected around the electrodes of the Main Unit. Absorb with a soft cloth.</li> <li>Check for dry residual medication. Disinfect appropriate components and accessories.</li> <li>Replace same type batteries with new alkaline or charged NiMH batteries.</li> <li>Low nebulization rate may indicate it's time to replace the Mesh Cap.</li> </ul>
<p>The unit is on, however it nebulizes weakly or is taking to long for treatment.</p>	<ul style="list-style-type: none"> <li>Make sure the Medication Bottle is properly installed.</li> <li>Follow directions for cleaning after each use.</li> <li>Follow directions for disinfecting.</li> <li>Replace batteries per installation instructions.</li> <li>Nebulization rate may vary based on the medication used.</li> </ul>

# WARRANTY INFORMATION

## LIMITED WARRANTIES

Your NE-U22V/NE-U22VAC Nebulizer, excluding the mesh cap and accessories, is warranted to be free from defects in materials and workmanship appearing within 2 years, when used in accordance with the instructions provided with your device. The above warranty extends only to the original retail purchaser.

We will, at our option, repair or replace without charge your Omron Nebulizer. Repair or replacement is our only responsibility and your only remedy under the above warranties.

To obtain warranty service, ship the device and all of the components, together with proof of purchase and a note explaining the problem with \$5.00 for return shipping and insurance to:

Omron Healthcare, Inc.  
Attn: Repair Department  
300 Lakeview Parkway  
Vernon Hills, IL 60061

Be sure to include the model number, your name and your phone number on the enclosed note and any other correspondence.

**ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

**OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.** Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights, which may vary from state to state.

## LIMITED WARRANTIES (OUTSIDE OF THE USA AND CANADA)

Should repair be needed during the warranty period, please contact the authorized distributor in your country.

### FOR CUSTOMER SERVICE (US and CANADA)

Visit our web site at:

[www.omronhealthcare.com](http://www.omronhealthcare.com)

Call toll free:

1-800-634-4350

# SPECIFICATIONS

Omron’s new Micro-Air Electronic nebulizer with V.M.T. (Vibrating Mesh Technology) offers the smallest, lightest, most portable electronic nebulizer to the patient. Powered by two “AA” batteries, V.M.T. a technology pioneered by Omron, creates a dense aerosol by the movement of medication through a metal alloy mesh.

Model:	Omron <b>NE-U22V/NE-U22VAC</b> Electronic Nebulizer with V.M.T.
Power Source:	DC 3V AC 120V 60 Hz (with AC Adapter, available separately)
Power Consumption:	1.5 W
Nebulization Rate:	0.25 mL/min minimum
Particle Size Range:	MMAD approximately 5µm
Medication Capacity:	7 mL maximum
Operating Temperature/Humidity:	10°C to 40°C/30 to 85% RH
Storage Temperature/Humidity:	-20°C to 60°C/30 to 95% RH
Vibration Frequency:	180 kHz
Dimensions:	38mm x 51mm x 104mm (l x w x h) 1.5in x 2.1in x 4.1in
Weight:	97 grams, approximately 3.4 oz
Battery:	2 “AA” Alkaline or NiMH Rechargeable (not included)
Battery Life:	4 Hours (continuous use)
Contents:	Main unit, medication bottle, carrying case, mouthpiece, mesh cap, mask and mouthpiece adapter, video and instruction manual

Specifications and appearance are subject to change without notice for purposes of improvement.

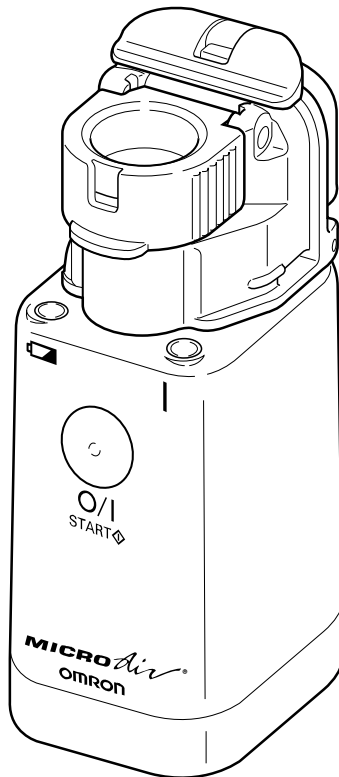
Distributed by:  
**OMRON HEALTHCARE, INC.**  
300 Lakeview Parkway  
Vernon Hills, Illinois 60061  
[www.omronhealthcare.com](http://www.omronhealthcare.com)  
Copyright © 2003 Omron Healthcare

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Copyright © 2003 OMRON Omron Healthcare, Inc.