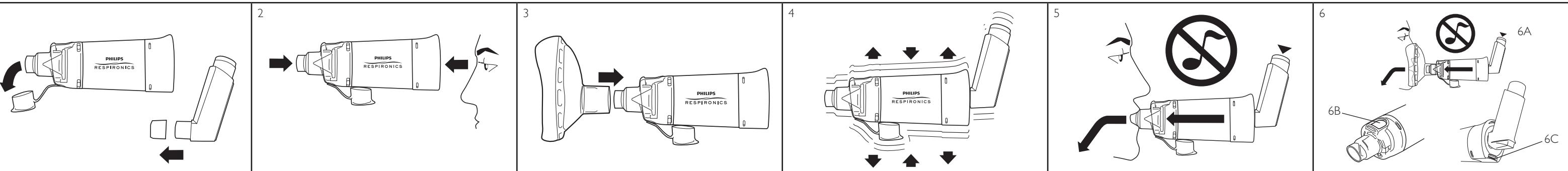
# OptiChamber Diamond

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Single patient use

### OptiChamber Diamond anti-static valved holding chamber

**ENGLISH Instructions for use**  
This device can be used right out of the package. Prior to use, read these instructions and those supplied with the pMDI. Always follow your physician's instructions.

Remove caps from both the pMDI and mouthpiece.

Prior to use examine the device for damage and missing parts. Remove any foreign objects or debris from device and optional mask. Do not use device if parts are damaged or missing. Contact Philips Respironics customer service if you have any questions or you need to replace OptiChamber Diamond.

When using OptiChamber Diamond with LiteTouch mask, place the mask onto the mouthpiece end and gently twist until it fits securely.

Insert the pMDI securely into the flexible opening of the end cap (adapter) then shake according to the pMDI instructions.

Place lips around the mouthpiece to form a tight seal. Exhale. Immediately actuate the pMDI once and begin to inhale slowly. Slow your inhalation if the inhalation flow alert sound is heard. Hold your breath for up to 8 seconds or as directed by your healthcare professional.

If using a mask, press and hold the mask against the face to create a seal. Actuate pMDI and maintain the seal for 5-6 breaths. If additional doses have been prescribed by your physician, repeat the above instructions as needed. (6A)  
Do not cover the exhaust valve during use. (6B)  
Do not block the inhalation high flow alert during use. (6C)

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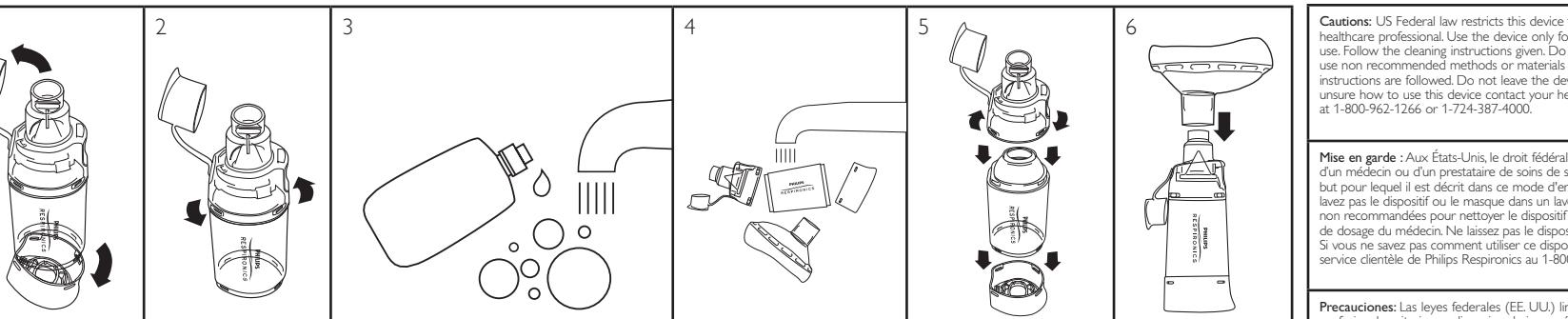
# OptiChamber Diamond

## Anti-static valved holding chamber

### ENGLISH

#### Cleaning instructions

It is recommended that the OptiChamber Diamond VHC and optional mask be washed once a week. Caution: Do not tamper with or remove the valves from your OptiChamber Diamond VHC during cleaning. Clean only as directed. Do not sterilize or boil.

Pull off protective cap. (A)  
Remove the end cap. (B)  
If using the LiteTouch mask, remove the mask.

Rotate the mouthpiece to disassemble.

**Device:** Agitate parts in warm soapy water with liquid detergent for a minute. Allow parts to soak for at least 10 minutes.  
**Mask:** Agitate in warm, soapy water for 2 minutes or soak for 10 minutes.

Rinse device and mask with clean running warm water and shake out excess moisture. Let all parts air dry thoroughly before reassembly.

To reassemble, place mouthpiece on chamber and rotate to lock. Reassemble the end cap onto the chamber.

If using a mask, replace onto the mouthpiece.

**Cautions:** US Federal law restricts this device to sale by or on the order of a physician or licensed healthcare professional. Use the device only for its intended use as described in these instructions for use. Follow the cleaning instructions given. Do not wash the device or mask in the dishwasher. Do not use non recommended methods or materials to clean the device or mask. Ensure physician's dosage instructions are followed. Do not leave the device and mask unattended with or around children. If unsure how to use this device contact your healthcare provider, or Philips Respironics customer service at 1-800-962-1266 or 1-724-387-4000.

**Indications for use:** The OptiChamber Diamond valved holding chamber device is intended to be used by patients who are under the care or treatment of a physician or licensed health care professional. The device is intended to be used by these patients to administer aerosolized medication from most Pressurized Metered Dose Inhalers. The intended environments for use include the home, hospitals and clinics. For single patient use. Recommended patient population: OptiChamber Diamond: age 5 and up • OptiChamber Diamond with Small LiteTouch mask: 0 to 18 months • OptiChamber Diamond with Medium LiteTouch mask: 1 to 5 years • OptiChamber Diamond with Large Lite Touch mask: 5 years +

**Important:** SAVE these instructions and store in a safe place. Follow manufacturer's instructions that come with pMDI inhaler. This device and the optional mask do not contain natural rubber latex material. When not in use store the device and optional mask in an appropriately clean place, like in a clean polybag.  
Storage/Operating Conditions: 0° F to 140° F (-18° C - 60° C)  
Humidity: 95%

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trademarks of Koninklijke Philips Electronics N.V. and its affiliates. All rights reserved.
Respirronics New Jersey, Inc.