



Reliable aerosol delivery

Instructions for using OptiChamber Diamond valved holding chamber

PHILIPS

RESPIRONICS

sense **and** simplicity

OptiChamber Diamond helps you get your inhaler medicine into your lungs, where it works

Please follow these instructions carefully, and also read the instructions for use inside the OptiChamber Diamond box.



Remove the caps from the inhaler and the Diamond chamber. Look for and remove any foreign objects in the chamber.



Insert the mouthpiece of your inhaler securely into the back of the Diamond chamber.



If a mask is needed, place the mask onto the mouthpiece end and gently twist until it fits securely.



Shake the chamber and inhaler 4 – 5 times.



Place your lips around the mouthpiece to form a tight seal. Exhale. Spray one puff of medicine into the chamber and immediately begin to inhale slowly, taking a full deep breath.*



If the flow alert 'whistle' sounds, breathe in more slowly. Hold your breath for up to 8 seconds and then breathe out.



Or if using a mask, press and hold the mask over the mouth and nose. Spray one puff of medicine into the chamber and breathe in and out for 6 breaths.*



The exhalation valve will open when the user breathes out, which can help you to count the breaths.

* If your doctor has instructed you to take more than one puff of medication, wait for one minute and then repeat steps 5 – 8.

PHILIPS

RESPIRONICS

sense and simplicity

**Philips Healthcare is part of
Royal Philips Electronics**

How to reach us

www.philips.com/healthcare

healthcare@philips.com

fax: +31 40 27 64 887

Asia

+852 2821 5888

Europe, Middle East, Africa

+49 7031 463 2254

Latin America

+55 11 2125 0744

North America

+1 425 487 7000

800 285 5585 (toll free, US only)

Respironics New Jersey, Inc.

5 Wood Hollow Road

Parsippany, NJ 07054 USA

Customer Service

+1 724 387 4000

800 345 6443 (toll free, US only)

Respironics Asia Pacific

+65 6298 1088

Respironics Australia

+61 (2) 9666 4444

Respironics Deutschland

+49 8152 93 06 0

Respironics Europe, Middle East, Africa

+33 1 47 52 30 00

Respironics France

+33 2 51 89 36 00

Respironics Italy

+39 039 203 1

Respironics Sweden

+46 8 120 45 900

Respironics Switzerland

+41 6 27 45 17 50

Respironics United Kingdom

+44 800 1300 845

www.philips.com/respironics

Please visit www.philips.com/optichamberdiamond to find out more



© 2011 Koninklijke Philips Electronics N.V. All rights are reserved.

Respironics, OptiChamber and LiteTouch are trademarks of Koninklijke Philips Electronics N.V. and its affiliates. All rights reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

PN 1096852 MCI 4104587 Issue 1 / bdc14354 * SEP 2011